

OF CORPULS3 AND CORPULS CPR



TEAMWORK TO PERFECTION SYNCHRONISATION OF CORPULS³ AND CORPULS CPR

We have had this vision for a long time: The perfect interaction between **corpuls3** and **corpuls cpr** during resuscitation. Raising the resuscitation processes to the next level of quality was our goal.

With its revolutionary modular design, the **corpuls3** stands out from other compact devices. It can be separated into the Monitoring Unit, Patient Box and Defibrillator/Pacer. Thanks to the synchronized therapy, stress amongst the team can be significantly reduced. **corpuls3** and **corpuls cpr** become one and when integrated into the team, this duo makes resuscitation even more efficient.

Hands-off time is significantly reduced* and the patient's chance of survival is increased – even under the most difficult conditions where space is at a premium.

mCPR mCPR Mode cont.

mCPR 57

°O

Manual

2 Charge

Monitor



SPECIFICATIONS CORPULS3

- Weight: only 6.5 kg (SLIM, basic configuration)
- Extremely high dust and splash protection (IP55)
- **Operation range**: -20 °C to +55 °C (basic functions: ECG monitoring, defibrillation)
- DIN EN 1789
- Aviation Standard RTCA DO 160 G
- US Military Standard MIL-STD-810G

• Transflective 8.4" Display, with opt. touch screen

Scorpuls

• Control of the **corpuls cpr** via Bluetooth

60 120 41

- Integrated 4G modem and WLAN or LAN connection for data transfer/telemedicine
- ECGmax and CEB[®] the 22-lead ECG solution from corpuls



* Refer to the chapter "Abstract Study"

SPECIFICATIONS corpuls cpr

- Weight: 5.5 kg (arm with battery and stamp)
- Compression depth: 2–6 cm
- Frequency: 80–120/ min
- Therapy mode: 30:2 | 15:2 | continuous
- Bluetooth and NFC
- **Intuitive user interface:** Therapy start/stop key with alarm and 4 softkeys

When designing the corpuls **cpr**, we focused on two things: A seamless rescue chain from pre-clinical, to air rescue to hospital, as well as the shortest possible hands-off time.

This is the reason why the **corpuls cpr** arm can be aligned and fixed over the patient with just one movement in just a few seconds. During the therapy the **corpuls cpr** checks the position of the stamp after each ventilation break or after 100 compressions when in continuous mode. If the thorax has collapsed, the **corpuls cpr** automatically corrects the distance between the stamp and the thorax. Thus ensuring that the set compression depth is always achieved.

With the three different boards made of radiolucent carbon, the user is optimally equipped for every mission.



Monitor









d

Customizable therapy parameters
Battery running time: up to 90 min
Radiolucent boards
Quick & safe adjustment
Treatment of children from 8 years of age, pregnant women and obese patients possible





Children

Pregnant C

Obese



CONNECTION IN 3 CLICKS

During synchronized therapy, the **corpuls3** is not just the remote control for the **corpuls cpr** - although that is also possible. The **corpuls cpr** has been perfectly integrated into the AED and manual mode of the **corpuls3**. Therefore giving the rescue team the ability to treat the patient in accordance with the current guideline recommendations. The corpuls3 shows all the required information on its display.

Ø

P



Connect key to connect with the **corpuls cpr** via Bluetooth

∦

Bluetooth symbol symbolizes a Bluetooth connection to the **corpuls cpr**



Pie chart with corpuls cpr symbol indicates a connection with the **corpuls cpr** and shows the 2 minute resuscitation cycle

Start/Stop key to start and stop the mechanical chest compressions by the corpuls cpr

Cont. CPF

Reset the time in the pie chart, if not defibrillated

FULL CONTROL – AT ANY TIME

Once connected to each other, it is no longer necessary to to operate the **corpuls cpr** during a resuscitation. The **corpuls cpr** is completely automatically controlled by the corpuls3, regardless of whether it is in AED or manual



mode. All of the **corpuls cpr** therapy parameters are displayed on the monitor of the corpuls3.

If you would like to change the settings manually or stop or start the **corpuls cpr**, this is also possible.



INTELLIGENT CONTROL WITH THE CORPULS3

The intelligent control of the corpuls cpr by the corpuls3 during a resuscitation follows the current guideline recommendations - depending on the mode (AED, Manual-ERC, Manual-AHA). In each of the modes, the entire rhythm analysis is controlled via

STARTING SITUATION:

The **corpuls cpr** is positioned on the patient and is performing chest compressions. There is a Bluetooth connection to a corpuls3.

AED MODE

- 1. Analyse key interrupts the corpuls cpr for the AED analysis and then automatically continues the therapy by corpuls cpr.
- 2. If the rhythm is shockable, the defibrillator is charged and automatic preShock CPR is performed.
- 3. The Shock key interrupts the corpuls cpr for shock delivery.
- 4. If the **rhythm is not shockable**, the **corpuls cpr** therapy is automatically continued

MANUAL 1 (procedure according to ERC)

- 1. The stop mCPR key on the corpuls3 interrupts the corpuls cpr for manual rhythm control.
- 2. The Start mCPR key resumes chest compressions/ preShock CPR.
- 3. If the **rhythm is shockable**, the **Shock key** interrupts the **corpuls cpr** for shock delivery. If the rhythm is not shockable, the Start mCPR key continues the therapy by the **corpuls cpr**.

MANUAL 2 (procedure according to AHA)

- 1. Charge key loads the defibrillator.
- 2. The Stop mCPR key on corpuls3 interrupts the corpuls cpr for manual rhythm control.
- 3. If the rhythm is shockable, the Shock key interrupts the **corpuls cpr** for shock delivery.
- 4. Start mCPR key resumes the therapy by the corpuls cpr.

the **corpuls3** as the central operating unit. The user only needs to monitor the correct position of the stamp on the **corpuls cpr**. All control commands for the user take place wirelessly between corpuls3 and corpuls cpr.









PRESENTATION OF THE CPR SYNCHRONISATION AT THE CIS 2020

CORPULS INNOVATION SUMMIT

INFOTAINMENT

DIGITAL | LIVE | TALK |

Corpuls has been a leader in innovative medical technology for years. Even during the coronavirus pandemic, our company from southern Germany has come up with unique ideas – such as a panel discussion about new innovations, produced in-house and streamed live.

An interview by Managing Director Dr. Christian Klimmer from this discussion:

What was the idea behind synchronized therapy?

When developing the **corpuls cpr**, we already had the vision of connecting the device to the **corpuls3** via Bluetooth. We knew from the **corpuls3** that fundamentally this would be possible and safe. The possibilities were obvious from the beginning: The top priority was an almost automated and synchronized resuscitation through the intelligent interaction of these two devices.

The corpuls cpr has been on the market for a while. Why did it take so long to implement this new feature?

At the time of product launch, the **corpuls cpr** set new standards in mechanical resuscitation and it was extremely challenging in development. We are very proud that we mastered those various challenges. However, in order to synchronize the two devices, extensive feasibility and

usability studies were required. It just takes a little longer in this regulated field we are in.

What do you think the five biggest advantages of synchronizing the corpuls³ with the corpuls cpr are? It's a shame that I can only mention five advantages, because I actually see a lot more. But let me get started. First: The hands-off time during resuscitation is significantly reduced. Second: The synchronisation frees up time and resources for the rescuers, which can be used for other purposes. Third: There is only one user interface for the user. This reduces the potential for error and the user has full control over the two devices at all times. Fourth: As the corpuls cpr also transmits all the mission data to the corpuls³, a data record is created for debriefing that comprehensively shows all aspects of the mission. In turn improving the quality of subsequent missions. Fifth: I also see an improvement in the occupational safety of the users, especially when transporting the patient in a vehicle. The removable monitor allows the **corpuls cpr** to be controlled remotely, without the user having to take off their seat belt.

Do you think synchronized therapy will be adopted by the market?

Yes. I'm very sure of that. Ever since we first introduced the **corpuls cpr** we have been asked whether it can be connected to the **corpuls3**. We are convinced that with this new function, we can now offer all the customers who wanted this connection all the options that they had in mind.

How have you reacted to the coronavirus pandemic?

Fortunately, we had an IT structure from the beginning that made it possible for employees to immediately start working from their homes. Our manufacturing department has achieved incredible things and built devices from early in the morning until late at night so that we could fulfill all orders asap. We also have a comprehensive hygiene concept for the employees in the office.

How did you carry out the shock synchronisation study under these circumstances?

That was definitely a problem. Due to the social distancing restrictions and lock-down measures, we were unable to conduct the study with our partner in Munich as planned. But we are fortunate that we could fall back on the around 50 employees with (emergency) medical training within our company. The previously mentioned hygiene concept made it possible to carry out the shock synchronisation study here in house.

WHAT IS "CIS"?

corpuls presents the latest developments to sales partners and the public at the **corpuls innovation summit** (CIS). This year however, due to the coronavirus pandemic, the event took place as a worldwide streaming event. Customers and those interested were able to tune in live to see the new products in action and ask the developers questions directly.



Which software version do I need for the synchronized therapy?

Any **corpuls3** that already has software version 4.0 has the feature on board. The **corpuls cpr** is compatible from software version 1.2. Older devices can be upgraded of course.

Do I need training to be able to use the new functions?

As always, the operation is very intuitive, but training is mandatory. Synchronized therapy must not be used without the instruction from Application Specialists or our partners.

Is my corpuls³/corpuls cpr Bluetooth compatible? Can my corpuls³ be retrofitted?

Yes, all our devices are compatible. The **corpuls cpr** Bluetooth function can be activated. If the **corpuls3** does not have Bluetooth, this function must be retrofitted.

What happens if the connection is lost?

If both devices lose their connection, the **corpuls cpr** continues the current therapy and the connection can be re-established. It is also no problem to operate another **corpuls** device nearby.

Does shock synchronisation cost anything?

No, there are no additional costs after set up.

What therapy parameters can be set by the corpuls³?

All the parameters that are adjustable on the **corpuls cpr** can be set by the **corpuls3**.













DATA ALWAYS IN VIEW

With the combination of corpuls.manager, the management software solution for corpuls devices, follow-up becomes child's play.



corpuls cpr

With just a few clicks, all curves of the corpuls3 (e.g. ECG, CO2 or IBP) can be visually represented on a computer together with the corpuls cpr compression data. Thus, past missions can be analysed and future operations can be optimized. corpuls.manager enables an overall picture of one or more resuscitation missions in unprecedented detail, without much effort. Users who already work with corpuls. manager or its predecessor corpuls.web REVIEW will not need any adjustment period. The merging of the data from



alte 🍌 REKS 🖪 Ensitzeniari 📈 Tends

In REVIEW by corpuls.manager you can clearly see that when the shock key is pressed, the corpuls cpr automatically stops, the shock is triggered and then the CPR is automatically continued.

Manna

both devices occurs completely automatically in the corpuls3 as soon as the device connects to the **corpuls cpr**. When the corpuls³ establishes a connection to the corpuls.manager server via cellular network or WLAN, the mission data is automatically uploaded.

Then, in corpuls.manager you can specifically filter for relevant missions and carry out statistical analysis based on your needs.

GUIDELINE BASED

Guidelines have become an indispensable part of modern medicine. We have aligned the synchronisation between corpuls3 and corpuls cpr to the basis of the current guideline recommendations according to Monsieurs et al. 2015, as high-guality chest compressions increase the patient's chance of survival:

corpuls®

- At least 5 cm compression depth (maximum 6 cm)
- Compression rate of 100-120 compressions/min
- Complete recoil of the chest in between compressions
- After the application of a defibrillator, defibrillation should not be delayed any further
- Chest compressions should be continued while the defibrillator is charging
- <10 sec interruption before and after shock delivery

- The chest compressions should only be minimally interrupted, even for defibrillation attempts
- Chest compression devices are recommended when it is not practical to perform good manual compressions or the safety of the rescuer is compromised
- A pre-shock pause of 5–10 seconds already reduces the probability of a successful shock

• 19

ABSTRACT STUDY

The ERC guidelines recommend the shortest possible compression interruptions during cardiopulmonary resuscitation. Before and after the shock delivery, chest compressions should be interrupted for a maximum of 10 seconds. A peri-shock pause (the compression break before and after defibrillation) of a maximum of 10 seconds has been scientifically proven to be associated with a higher survival rate (Cheskes et al. 2014; Sell et al. 2010). According to the ERC guidelines, the entire



The reduction in the compression pause is particularly evident during the perishock pause. On average, the perishock pause in synchronized mode of 2.00 \pm 0.32 s is 38.23% shorter than



References

Cheskes, Sheldon; Schmicker, Robert H.; Verbeek, P. Richard; Salcido, David D.; Brown, Siobhan P.; Brooks, Steven et al. (2014): The impact of peri-shock pause on survival from out-of-hospital shockable cardiac arrest during the Resuscitation Outcomes Consortium PRIMED trial. In: Resuscitation 85, S. 336-342, DOI: 10.1016/j.resuscitation.2013.10.014 Kleinman, Monica E.; Brennan, Erin E.; Goldberger, Zachary D.; Swor, Robert A.; Terry, Mark: Bobrow, Bentley J. et al. (2015): Part 5. Adult Basic Life Support and Cardiopulmonary Resuscitation Ouality Circulation. 2015;132[suppl 2]:S414–S435. In: Circulation 132 (18 suppl 2), S. S414-S435. DOI: 10.1161/CIR.00000000000259. Monsieurs, Koenraad G.; Nolan, Jerry P.; Bossaert, Leo L.; Greif, Robert; Maconochie, Ian K.; Nikolaou, Nikolaos I. et al. (2015): European Resuscitation Council Guidelines for Resuscitation 2015: Section 1 Executive summary. Resuscitation 95 (2015) 1-80. In: Resuscitation 95, S. 1-80. DOI: 10.1016/j.resuscitation.2015.07.038. Nolan, Jerry P.; Soar, Jasmeet; Zideman, David A.; Biarent, Dominique; Bossaert, Leo L.; Deakin, Charles D. et al. (2010): European Resuscitation Council Guidelines for Resuscitation 2010. Section 1 Executive Summary, In: Resuscitation 81 (10), S. 1219–1276, DOI: 10.1016/j.resuscitation.2010.08.021, Perkins, Gavin D.; Olasveengen, Theresa M.; Maconochie, Ian; Soar, Jasmeet; Wyllie, Jonathan; Lockey, Robert Greif Andrew et al. (2017); ERC 2017 Guidelines Update. In; Resuscitation, DOI; 10.1016/i. resuscitation 2017 12 007

Sell, Rebecca E.; Sarno, Renee; Lawrence, Brenna; Castillo, Edward M.; Fisher, Roger; Brainard, Criss et al. (2010): Minimizing pre- and post-defibrillation pauses increases the likelihood of return of sponta neous circulation (ROSC). In: Resuscitation 81 (7), S. 822-825. DOI: 10.1016/j.resuscitation.2010.03.013.

manual defibrillation process should be possible with a compression break of less than 5 seconds (Monsieurs et al. 2015). When using the **corpuls3** in combination with the **corpuls cpr** in an in-house study, these recommendations could be noticeably undercut. With the synchronisation of the corpuls3 and the **corpuls cpr**, this endeavour could be further increased. With synchronized device operation, the hands-off time was reduced by 16.08%

> Illustration 1: Representation of the hands-off time of our study comparing synchronized and disconnected device operation of the corpuls3 and corpuls cpr (Significance P < 0.001: ***: P < 0.05: *)

the peri-shock pause in disconnected mode of 3.24 ± 0.71 s. This saving can be a valuable gain in time with the desired break being a maximum of 5 seconds.

Illustration 2:

When comparing the peri-shock pauses, the significant minimization of the pause duration with synchronisation compared to disconnected device operation of corpuls3 and corpuls cpr becomes particularly clear. (Significance P < 0.001: ***)



corpuls simulation TRAIN BEFORE YOU FIGHT

There is often a big difference between theoretical scenarios and actual missions. With **corpuls simulation** we bring completely realistic scenarios to your training. Just as pilots training in a flight simulator are thrown surprise complications, rescuers training with **corpuls simulation** have to overcome unforeseen situations. The result is a team that is optimally prepared for every scenario and therefore, a significantly better quality of care for the patient.

↓

corpuls®



SYNCHRONISATION THERAPY

Training not only means using existing knowledge, but also learning new techniques – that includes synchronized therapy with **corpuls3** and **corpuls cpr**. Thanks to the Bluetooth connection from **corpuls simulation**, this feature can now be trained realistically at any time. All usage scenarios and benefits can be optimally taught for use in the field: simply connect **corpuls simulation** with the **corpuls cpr** (from software version 1.2) and take your training to a whole new level.

THE FACTS

- All the functions of the original device
- Realistic parameters and curves
- Status changes through events
- Scenario editor
- Simulation consumables
- Original accessories
- The latest device software for the respective
 medical products
- Multiple languages available
- Safe training
- corpuls simulation community
- Bluetooth connection to corpuls cpr

FOCUS - ON - PATIENTS



* Using the combination of **corpuls simulation** and **corpuls cpr** only on training manikins

THE ECONOMICAL SOLUTION

Training as with a high-end simulation manikin – but for a fraction of the price.

Display of:

- Different ECG rhythms
- Perfusion dependent SpO₂ curve
- NIBP parameters or IBP with curve
- etCO₂ parameters and waveforms
- CPR Feedback display

corpuls simulation in action – Find the video here.













For over 35 years, **corpuls**[®] has developed and produced innovative high-end equipment for emergency and intensive care medicine. Today, in our headquarters in Kaufering, over 300 hearts each beat around 60,000 times every work day while aspiring to meet the high standards of rescue workers from over 70 countries around the world.

corpuls defibrillators, patient monitoring systems and chest compression devices have set the standard since day one in the realisation of the most advanced insights in medical science, as well as in terms of innovation and ergonomics. Thus guaranteeing reliable and trusted aid in the fight for human lives.





Manufacturer

corpuls | GS Elektromedizinische Geräte G. Stemple GmbH Hauswiesenstraße 26 | 86916 Kaufering

Telephone+49 8191 65 722-0E-Mailinfo@corpuls.comWebwww.corpuls.world



Products may not be available in all markets as product availability depends on the regulatory and/or medical processes in individual markets. For availability please contact **info@corpuls.com**. Printing errors as well as construction and design modification subject to change. All mentioned product names are registered trademarks of the respective owners. Art.-Nr. 76193.34020 Vers. 1.0 (10/20)

C E 3797

VectraCor | 785 Totowa Road, Suite 100 | Totowa NJ 07512 | USA

